




User's Manual

Stairlift Model HW 10

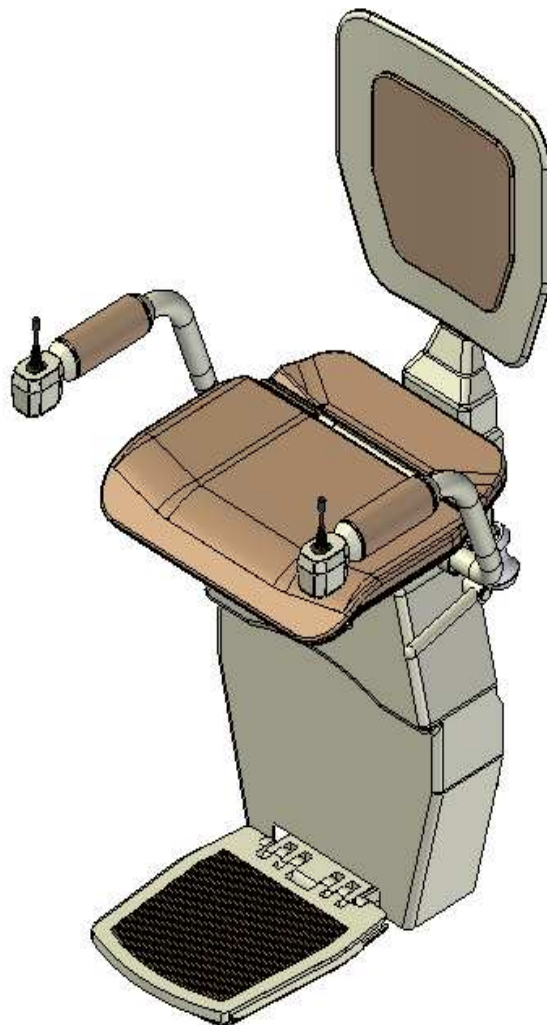

Contents

General Description	3
Instructions	
- Lift	4
- Remote Control	5
Detailed View	6
Visual Indication	7
Safety	7
Maintenance	
- By User	8
- By the Technican	8
Warranty and Service	9

1 General Description

The HAWLE stairlift model HW 10 has been designed for the conveyance of adults with locomotive problems or elderly persons in the interior of buildings with one or more floors. The HAWLE HW 10 is a complex lift construction which can be installed both outside and inside of the stairway. In order to save space, the lift can be folded on its parking position. The distance between rail and wall is more narrow when compared to other manufacturers' stairlifts, and the maximum angle of inclination is higher than any other stairlift system. The special concept and the high quality construction allow the travel over most barriers and hindrances in your home with two, three, or even more floors. Using the remote control, the lift can be sent to any position you like. We can offer you the best price at the highest quality by designing and producing the HAWLE stairlift ourselves. After substantial tests and research, the life of each lift was determined to be 15 years while using it between 800-1500 times per year and having the necessary service performed at routine intervals. We can offer you a guarantee for the product covering 2 years on parts and 5 years on the drive-train. For your safety this HAWLE stairlift model has been furnished with all the required safety devices and functions, such as emergency sensors. We can offer special equipment like the swivel seat for example as well.

Important: Please make sure that the blue checklight on the armrest of your HAWLE stairlift is glowing when the lift stops in the parking position. It shows the correct charging condition. Otherwise the batteries could be damaged, which are **not** included in the warranty. (Estimated life of the batteries is 5 years if used properly). If you have to disconnect the main power supply, please make sure to turn off the lift system first. Otherwise, the batteries will lose their capacity and become damaged.


Best regardsž

HAWLE Team

2 Instructions

2.1 The lift

Please tilt down the armrests and the seat before using the lift. (Fig. 3: # 2,3,7)
By pulling the lever (6) tilt down the foot support. Now your HAWLE stairlift is ready to ride. You can start the run by using the joystick (1) on your armrest. Press the joystick in the direction you want to ride and keep it pressed until the lift reaches its final position and stops automatically. If you have a powered swivel seat, keep the swivel seat key pushed (4) and the seat of your HAWLE stairlift will turn and allow you a comfortable and safe mounting and dismounting. After reaching the final position, use the swivel seat switch to turn the seat to its standard (parked) position. Now it is possible to tilt up the foot support, seat, and armrests in order to save space.

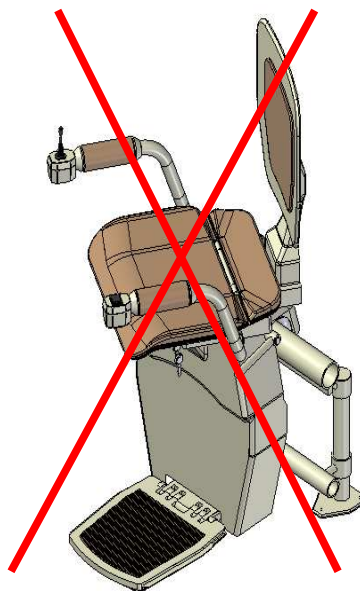

Incorrect

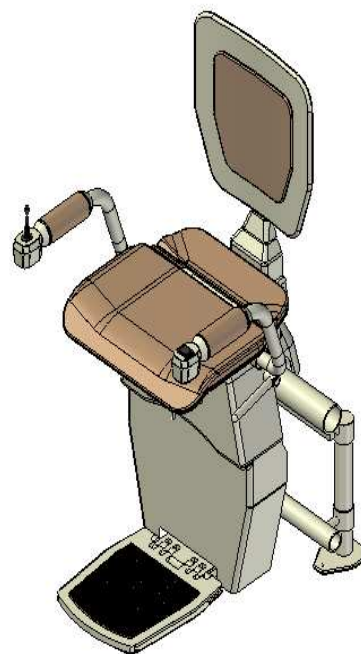


Fig. 1

Correct

2.2 Remote Control

To move your Hawle stairlift from its parked position using your remote, just press the button for the desired direction on your remote control. Keep this button pressed until the lift reaches its final position and stops automatically.

To open the battery compartment, pull down the cover with the Logo. Now you can easily replace the battery.

Kind of Battery: 9V Type: 6F22 or 6LR61



Fig. 2

Detailed Overview

1. Joystick Control
Direction
Lift Up - Down

8. Blue LED
Parking

7.5 fa fYgh
@Zh

2. Armrest
Fight

3. Seat

6. : cclfYgh
Lever

- "; fYYb @ 8
fA Ujb Gk JHW L

4. Swivel
Seat
Control
(Optional)

5. Foot support

Fig. 3

4 Visual Indication

Green 7 hecklight

The green LED (Fig.3 (9)) indicates the function of your HAWLE stairlift. It keeps shining if the contact key (optional) is inserted and the main switch is turned "ON/EIN." The contact key and the main switch can be found on the lower frontside below the seat of your HAWLE stairlift. A flashing LED indicates that the batteries need to be charged. If the LED stops shining, the batteries have lost their capacity or the power supply was disconnected.

Blue 7 hecklight

The blue LED on the armrest of your HAWLE stairlift indicates that the driving unit reached its correct park and charging position.

5 Safety

- Please check the correct position of armrest and seat.
- Make sure to remove any objects that can obstruct the lift's travel.
- Place your arms on the armrest and your feet on the foot support to operate the lift correctly.

Please avoid...

- Standing on the lift while it is traveling.
- Carrying more than one person at a time.
- Using the lift during a fire.
- Transporting bulky and flammable objects.

6 Maintenance

6.1 By I ser

To facilitate a long lifetime without malfunctions of your HAWLE stairlift, the HAWLE stairlift needs maintenance. It is our recommendation that your lift is serviced every 12 months by the Hawle approved dealer that installed your lift. We do suggest cleaning the rail with a moist towel dampened with water if it becomes dirty.

6.2 By Hechnican

All the relevant mechanic and electric safety equipment will be inspected and exchanged (if necessary) by our service team and technicians.

7 Warranty and Service

As the manufacturer of your HAWLE stairlift we ensure the product functions properly. The warranty time is 2 years on parts 5 years on the drive-train. It begins once the lift has been installed.

The warranty does not cover malfunctions due to improper handling.

To prevent damages resulting from a malfunction while covered by the warranty, the owner is required to inform the HAWLE support **immediately!**

Support Line:

Please have your **client code** ready